AUSTRIA
Bauerfeind Ges.m.b.H.
Hainburger Straße 33
1030 Wien
P +43 (0)800 4430-130
F +43 (0)800 4430-131
E info@bauerfeind.at

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 english

Dear Customer,

thank you for choosing a Bauerfeind product.

We work to improve the medical effectiveness of our products every day – because your health is very important to us. Please read and observe these **instructions for use** carefully. If you have any questions, please contact your doctor or medical retailer.

Intended purpose

The MalleoLoc L3 is a medical device. It is an orthosis¹ for the lateral stabilization of the ankle – with elements that can be removed in three stages.

Discuss removal or the time of possible removal with your physician in advance.

 **During the first fitting of the MalleoLoc L3, the support must be individually adapted to the patient by a trained specialist². This is the only way to guarantee the full effect and ideal wearing comfort of the orthosis.**

Indications

- Non-surgical treatment of sprains and ruptured ligaments
- Acute and chronic capsular ligament instabilities
- Prevention of chronic ligament instability
- Post-operative rehabilitation / protection, e.g. after suture repair / ligament reconstruction
- Post-traumatic / post-operative irritation

Risks of using this product

 **Caution³**

Please closely observe the specifications in these instructions for use and the instructions given by the specialist.

- The MalleoLoc L3 must only be worn in accordance with the specifications contained in these instructions for use, and the areas of application listed.
- The use in combination with other products, e.g. when wearing compression stockings as part of compression therapy, must be discussed beforehand with your physician.
- No product liability is accepted in the event of improper use.

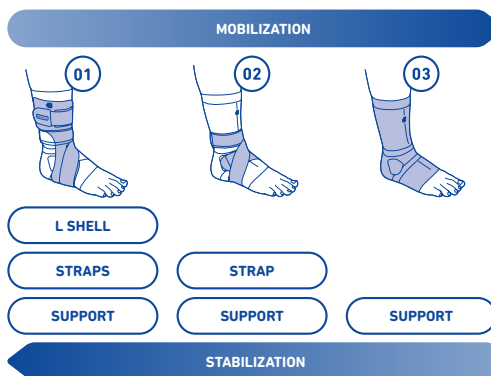
- Do not make any alterations to the product. Failure to comply with this requirement may adversely affect product performance, thus canceling any product liability.
- Side effects involving the body as a whole have not been reported to date.
- Correct usage / fitting is assumed. Do not fit the MalleoLoc L3 too tightly as this could lead to local pressure symptoms. In rare cases, constriction of blood vessels and nerves may occur. If this happens, you should loosen the straps of the MalleoLoc L3 and have the size of your MalleoLoc L3 checked, if necessary.
- If you notice any unusual changes or your symptoms get worse, stop using the MalleoLoc L3 and please contact your physician without delay.
- Caution: When loosening the straps or tensioning and fastening systems, or when removing the product or adjusting configured limitations of movement, there is not sufficient support / sufficient protection of the body part requiring treatment. In this case, particularly avoid placing any strain on this body part.
- Please check the floor and other walking surfaces to make sure they are non-slip before taking your first steps. Only run with extreme caution using short steps, remaining conscious of your ankle injury. Avoid slipping, as this could cause serious secondary injuries.
- If you would like to use your Bauerfeind product due to acute symptoms / injuries, make sure to obtain and follow the advice of a medical professional before first using the product. Restrictions may exist given the specific indication which the specialist retailer can clarify. In particular, the ability to operate vehicles, other means of transport or machinery may be limited. In case of doubt, we recommend temporarily refraining from the activities mentioned above.
- Do not allow the product to come into contact with ointments, lotions, or substances containing grease or acids.

 Contains natural rubber latex (Elastodiene) which can cause skin irritation in persons with an allergy. Take off the support if you notice any allergic reaction.

Contraindications

No clinically significant hypersensitive reactions have been reported to date. If you

Treatment levels

have any of the following conditions, such aids should only be fitted and worn after consultation with your physician:

- Skin disorders /injuries in the part of the body being treated, particularly if inflammation is present. Likewise, any raised scars with swelling, redness, and excessive heat build-up.
- Impaired sensation and movement disorders of the legs / feet (e.g. diabetes mellitus)
- Impaired lymph drainage, including soft tissue swellings of uncertain origin in parts of the body where the orthosis is not worn.

Application instructions

Putting on the MalleoLoc L3

Below you will find a detailed description of how put on and take off the product during Treatment Levels 1, 2, and 3:

Treatment Level 01

1. Sit down to put on the orthosis.
2. First, put on the support. Fully open the zip, and slip your foot into the support. 1 Make sure the dark gray stripe on the knitted fabric runs down the front center of the foot. Close the zip. It should be positioned between the Achilles tendon and the medial malleolus. 2 3
3. Now loosen the upper and lower strap on the plastic shell. The lower strap remains threaded into the slot on the sole. Loosen the lower strap so far that you can easily put your foot through the loop you created.
4. Place your foot on the floor at a 90° angle to your lower leg, as this is the best position for fitting the orthosis. With the support fitted, slide the foot into the orthosis between the strap loop and the shell, 4 so that the shell sits laterally on the foot in front of the lateral malleolus.
5. Using one hand, press the shell onto the foot so that it sits flat against it. 5 Use the other hand to guide the lower strap diagonally across the instep by pulling from the outside of the foot (from the shell). Fasten the strap at the lower Velcro triangle. During this process, the side of the strap with the blue velour must be attached to the blue plastic shell. 6
6. Wind the upper strap around the base of the calf by pulling the strap slightly, 7 then fasten it. 8
7. Now guide the lower strap above the medial malleolus and around the back of the Achilles tendon, 9 returning it to the front of the foot of the affected ankle. 10 Fasten the end of the strap on the strap (the Velcro area depends on the calf circumference).
8. Tighten both straps again, if necessary. 11

Treatment Level 02

1. In order to remove the orthosis for the 2nd Treatment Level, open the lower strap of the MalleoLoc L3 first. Remove the lower strap from the slot on the sole. Loosen the Velcro fastening that fixes the lower strap to the plastic shell. Keep the lower strap close by.
2. Store the plastic shell including the upper strap with the instructions for use in case you want to add it again later.
3. Sit down to put on the support.
4. First, put on the support. Fully open the zip, and slip your foot into the support. 1 Make sure the dark gray stripe on the knitted fabric runs down the front center of the

foot. Close the zip. It should be positioned between the Achilles tendon and the medial malleolus. 3

5. Take the prepared lower strap and use the transparent Velcro fastening to attach it to the blue, tear-shaped point on the blue section of the support facing towards the inside of the foot. 12 The tip of the blue, tear-shaped point indicates the direction of the strap.
6. Put the strap over the instep toward the inside of the foot. Then guide it below the foot 13 and, by pulling from the outside of the foot, diagonally across the instep. Now put the strap above the medial malleolus and around the back of the Achilles tendon, 14 returning it to the front of the foot of the affected ankle. 15 Fasten the end of the strap onto the strap. 16 The Velcro area depends on the foot circumference.

Treatment Level 03

1. For the 3rd Treatment Level of the MalleoLoc L3, you only need the support. Safely store the plastic shell, including the upper and lower straps, as well as the instructions for use.
2. Sit down to put on the support.
3. Pick up the support. Fully open the zip, and slip your foot into the support. 1 Make sure the dark gray stripe on the knitted fabric runs down the front center of the foot. Close the zip. It should be positioned between the Achilles tendon and the medial malleolus. 3

Taking off the MalleoLoc L3

1. Open the two Velcro straps.
2. Fully open the zip.
3. Remove your foot from the orthosis / the support.

Cleaning instructions

The sewn-in label of the MalleoLoc L3, which contains information on the product name, manufacturer, washing instructions and CE mark, is located on the upper edge of the support.

Note: Never expose the MalleoLoc L3 to direct heat (e.g. radiators, sunlight, in your car, etc.)! This may damage the material, which can impair the effectiveness of the MalleoLoc L3.

To clean the MalleoLoc L3, fasten the Velcro ends back onto the straps to avoid any damage. Clean the plastic shell by wiping it with a damp cloth.

The MalleoLoc L3 support can be washed at 30 °C by hand, or separately in the washing machine with liquid detergent. We recommend that you use a lingerie bag. Air dry the support. Do not dry it in a tumble-dryer because this can damage the knitted fabric.

Please wash your support regularly so that the compression of the knitted fabric can be retained.

Part of the body this product is used for

In accordance with the indications (ankle).

Maintenance instructions

If handled and cared for correctly, the product is practically maintenance-free.

Assembly and fitting instructions

The MalleoLoc L3 is supplied ready for use.

Assembling the MalleoLoc L3 after cleaning:

Fitting the straps:

The example below is for fitting the straps for the right side MalleoLoc L3.

Place the plastic shell in front of you with the sole pointing toward you, and the outside pointing up. Take the upper, shorter strap and position it in front of you with the blue velour pointing up. Thread the transparent Velcro end into the right-hand slot of the upper three slots, and back to the outside through the slot immediately adjacent. Attach it to the strap. Now take the other strap end, guide it toward the left in the direction of the left slot, and push it through the plastic shell from the outside.

Take the lower, longer strap and position it in front of you with the blue velour pointing down. Insert the transparent Velcro end in the slot in the middle of the orthosis from the right-hand side and from above, and attach it to the strap on the right. Take the other strap end and insert it into the opening in the sole.

Technical specifications / parameters

The MalleoLoc L3 is a removable orthosis for the ankle joint with two straps attached to it and an integrated support to apply effective compression to the affected ankle. The support and the stabilizing plastic shell are separate elements, the straps can be removed.

Notes on reuse

This product is intended to be fitted to one patient only.

Warranty

The statutory regulations of the country of purchase apply. Please first contact the retailer from whom you obtained the product directly in the event of a potential claim under the warranty. The product must be cleaned before submitting it in case of warranty claims. Warranty may be limited or excluded if the instructions on how to handle and care for the MalleoLoc L3 have not been observed. Warranty is excluded if:

- The product was not used according to the indication
- The instructions given by the specialist were not observed
- The product was modified arbitrarily

Disclaimer

Do not diagnose yourself or decide on which medication to take unless you are a medical specialist. Before first using our medical product, please seek the advice of a physician or trained specialist, as this is the only way to assess the effect of our product on your body and determine the potential risks of using this product due to your personal constitution. Follow the advice of this specialist as well as all instructions in this document or its online version, including any excerpts (as well as texts, images, graphics etc.).

If you still have doubts after your consultation with the specialist, please contact your physician or retailer, or contact us directly.

Duty to report

Due to regional legal regulations, you are required to immediately report any serious incident involving the use of this medical device to both the manufacturer and the responsible authori-

ty. Our contact details can be found on the back of this brochure.

Disposal

Once you have finished using the product, please dispose of it according to local specifications.

Material content

Polyketone (PK), Polyamide (PA), Elastodiene (ED), Polyurethane (PUR), Polyester (PES), Viscose (CV), Elastane (EL), Zinc (Zn), Steel, Polyvinyl chloride (PVC), Polyoxymethylene (POM)

 – Medical Device

 – DataMatrix Unique Device Identifier UDI

Version: 2024-02

¹Orthosis = orthopedic aid used to stabilize, relieve, immobilize, control, or correct the limbs or the torso

²A specialist is any person who is authorized, according to their state's regulations, to fit and provide instructions on the use of supports and orthoses.

³Note on the risk of personal injury (risk of injury, harm to health, and accidents) or damage to property (damage to the product)

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