# Dunimed Outdoor Rollator

*Dunimed Outdoor Rollator*

**GB:** This user manual is part of the delivery. It will be available to the user and will remain in the product when it is delivered. Due to technical progress, we reserve the right to change the version described in this manual.

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## GB: Product Parts

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**GB:** Indication and intended use - Aluminium Rollator

Do not exceed the maximum user weight limit of 136KG.

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## **GB: Picture Instruction**

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### **Step 1/Step 1:**

**GB:** Take out the rollator from carton, 4 Knob Screws are in the bag.

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## Step 2/Step 2:

**GB:** Open the packed rollator, put the front wheel and rear leg down, use knob screws to fix front wheel and rear leg **tightly**.

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### Step 3/Step 3:

**GB:** Hang the bag on the bag Hanger. And use the Velcro to fix the bag on the frame.

Press the silver locking pin to adjust the handle height to the suitable user height, the silver locking pin must be bounced out of the frame to lock.

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**GB:** If lock the brake, push down the brake lever, push up, walking again.

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**② Pull Up: Parking Released and Walking**

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**GB:** If with cane, put the cane into cane holder, then use Velcro to fix the top of cane.

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## **GB: Operation Instruction:**

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### **GB: Fold or unfold the rollator**

Place the rollator in front of you in the direction of travel and apply the parking brake.

#### **Unfold:**

With both hands, push the side tubes of the seat down and right down until the folding mechanism clicks into place. Do not touch under the seat to avoid injury from the locking mechanism. Loosen the front wheel locking bolts and then lower the wheels. Use the locking screws to lock the wheels in this position. Then please hang the shopping bag.

The handles of the rollator can be adjusted in height to fit the size of the user. To adjust the height, push in the locking pins on the left and right, then pull up the handle bar until the desired position is reached. There are eight positions available. The pins snap in automatically.

#### **Fold:**

Remove the shopping bag from the frame. Pull the locking band in the middle of the seat upwards - the rollator folds up. Loosen the knob screws of the front wheels and then fold the wheels upwards. Use the knob screws to lock the wheels in this position.

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**GB: Brake**

Service brake (when pushing): Pull the brake levers upwards.

Parking brake (when parking and using the seat function): Press the brake levers downwards.

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**GB: Climbing Aid**

Pressing the tip on the extended frame makes it easier to lift the steering wheels.

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**GB: Safety Instructions**

Always observe the minimum and maximum permissible height of the sliding tubes.

Do not hang on the handles - danger of tipping!

Use the service brake on steep roads (pull the brake levers up).

Use the rollator only on firm, level ground.

Never use the rollator to move when the rollator is folded! If you lean too much against the handles, the rollator can tilt backwards.

When folding and unfolding the rollator, clothes and body parts can be clamped.

Use for purposes other than walking, such as transporting heavy loads or kicking, is inappropriate and dangerous.

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### **GB: Cleaning**

Cleaning and disinfection should be carried out at regular intervals. Use a mild soapy solution without abrasives.

Standard methods can be used for disinfection. Do not use steam injection equipment or scrub or corrosive cleaners.

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