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DUN333

General guidelines

Take the time to thoroughly read the following instruction and utilize the product accordingly.

Ensure you retain this manual and pass it along when transferring the product.

Please be aware that this summary may not encompass every detail of all variations and deliberated steps. Feel free to reach out to us for additional information and assistance.

Warnings

Keep all parts and packaging materials (such as foil, plastic bags and foam) away from babies and children to avoid suffocation hazards.

Changes or modifications to the product without permission may compromise safety.

Do not use the product if you have a disability or are taking medication that may interfere with safe use of this product.

Clean the product with mild soap or household cleaner. Do not use organic solvents (such as acetone, MEK, lacquer thinner, lighter fluid, dry cleaning fluid, petrol or turpentine) that may adversely affect the product.

The Dunimed Handle for Seniors provides support for extra balance when getting up and sitting down in the shower or bath.

Please note that the Dunimed handle is not designed to support full body weight!

Notes

The Dunimed Handle for Seniors works on any non-porous, flat surface, including tiles, glass, acrylic and metal.

Please note: does not adhere to rough or uneven surfaces, including plasterboard! Tiles should be at least 4 x 4 inches (10 x 10 cm) and the product should not overlap joints.

The Dunimed Handle can be installed without drilling or using tools. The handle is not harmful to surfaces if attached and used correctly.

Before each use, check that the handle is firmly attached; reattach if necessary.

Installation steps

- 1. Surface preparation:** Make sure the surface is clean, smooth and non-porous. Clean with non-abrasive soap and water. Check that the surface is in good condition.
- 2. Check rubber suction cups:** Check that the rubber suction cups are smooth and not defective. Clean them with non-abrasive soap and water.
- 3. Positioning:** Make sure the suction cup is fully seated in the tile and does not cross any joints.
- 4. Fixation - first side:** With the tabs up, press the handle firmly onto the surface. This eliminates the air under the suction cup.
- 5. Locking:** Press the tabs all the way down for an airtight seal.
- 6. Repeat:** Repeat step 4 and step 5 on the other side of the handle.
- 7. Check:** Make sure the handle is securely in place.

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